

Companion add-on for manual wheelchairs: Airline Handling Guide

For proper airport personnel handling of Companion assistive mobility device.
Attach this document to device in a conspicuous area.

Owner's Contact Information:

Name: _____

Phone: _____

Device Serial #: _____



1. Turn-off Companion with the key (counterclockwise)
2. Remove key from keyhole.
3. Store key in a secure place.

Turning off the Companion and removing the key, physically disconnects the battery and the motor, preventing any accidental connection.

Battery

Type: Rechargeable Lithium-ion

Specs: 36 V; 270 Wh

The Companion rechargeable lithium-ion battery has been tested on UN Manual of Tests and Criteria Part 3 Subsection 38.3 and certified to meet the applicable ICAO, IATA and FAA requirements.